EQUIPMENT START UP AND SHUT DOWN QUICK REFERENCE CARD

DRX-1 SYSTEM

To start up the DRX-1 System:

1. Press the **On** switch on the Console. For systems that have an Uninterruptible Power Supply (UPS), press the **On** switch on the UPS.
2. When the software initializes, the system is ready for use.
3. Insert a fully charged battery into the detector.

To shut down the DRX-1 System:

1. Select the **Quick Menu** in the lower left corner of the screen.
2. Select **Shut Down**.
3. Turn **Off** the monitor. For systems that have a UPS, turn **Off** the UPS.
4. Turn **OFF** the Console.

DRX-MOBILE RETROFIT KIT

To start up the DRX-Mobile System:

1. Press the black **O/I** toggle switch labeled **Power On/Off** on the bottom edge of the computer/monitor.
2. When the software initializes, the system is ready for use.
3. Make sure a fully charged battery is inserted in the detector.

To shut down the DRX-Mobile System:

1. Select the **Quick Menu** in the lower left corner of the screen.
2. Select **Shut Down**.
3. Turn **Off** the monitor.
4. Select the **Shutdown/Power Off** radio button.
5. Select **OK**.
6. Select **Yes** to Are you sure you want to shut down?

You can remain in hold-on-prep for up to 15 seconds and the detector will function properly. In the event of an aborted exposure, the detector can acquire again in 4 sec.

DRX-EVOLUTION SYSTEM

To start up the complete DRX-Evolution System:

1. Switch **On** the Main Power at the Wall.
2. Verify that the **S1 Main** breaker and all system component switches are on.
3. Switch on the UPS by pressing the **On** button briefly. Verify that the **Line Power** LED is on.
4. Verify that the **Battery Charging** level indicates more than 10 %.
5. Power up the system by pressing the blue button on the Control Box, located next to the System Console.

To shut down the complete DRX-Evolution System:

1. Press the **blue** button on the Control Box, located next to the System Console.
2. Wait until the computer has shut down completely (approximately 4 minutes).
3. Switch **Off** the UPS by pressing the **Off** button.
4. For the LCD UPS model, press and hold the **Off** button until a second beep is heard.
5. Switch off the **Main Power** switch at the wall.

DRX-TRANSPORTABLE SYSTEM

To start up the DRX-Transportable System:

1. Make sure the USB cable is connected to the computer from the Console.
2. Press the **On** switch on the computer and monitor.
3. When the software initializes, select the **DRX** icon on the monitor screen.

To shut down the DRX-Transportable System:

1. Select the **Quick Menu** in the lower left corner of the screen.
2. Select **Shut Down**.
3. Turn **Off** the monitor.
4. Turn **Off** the UPS.

DRX-REVOLUTION MOBILE X-RAY SYSTEM

To start up the DRX-Revolution System:

This procedure assumes that the cart has been parked and docked, and has been plugged into a wall socket to recharge the batteries.

1. Carefully remove the power cord from the wall socket by holding the plug housing and pulling it from the power outlet. Tug slightly on the cord and slowly release it to start the rewind mechanism. Do not pull on the cord to remove it from the wall.
2. Touch the Tube Head monitor screen on the tube head assembly. The system “wakes up” and the Logon screen is displayed.
3. To log in, either enter your user name and password on the display or swipe your ID badge over the proximity reader.

To shut down the DRX-Revolution System:

1. Dock the boom and return the cart to its parking/recharging location.
2. Grab the plug at the cart and gently pull out enough of the power cord to place the plug fully into an AC voltage wall socket without stretching or pulling on the cord. After a predetermined time, the system automatically enters a screen saver mode, power is cut to the monitor light, and you will be automatically logged out.

- You will remain logged in until another radiographer logs in, or until you are automatically logged out by the system.

- If you swipe your badge while the primary monitor is still asleep, it will log you into the system and wake up the primary monitor. There is no need to wake up the primary monitor.

- If you disconnect the power cable, it may wake up the primary monitor. Then there is no need to touch the screen.

Classic/Elite and CR Classic Reader with Easy View display

To start up the CR System:

1. Press the power switch on the side of the Classic/Elite CR and the CR Classic.
2. Wait for the system to initialize.
If the **Power** LED light is not on, press the **Power** button on the computer.

To shut down the CR System:

On the Classic/Elite and CR Classic with Easy View display turn off the power switch on the left side of the reader.

For Classic/Elite systems without Easy View display, at the Main Menu, select the Quick Menu.

1. Touch **Shutdown/Power Off** and then select **OK**.
2. Turn off the power switch on the side of the reader.

MAX CR/CR 975 SYSTEM

To start up the Max CR/CR 975 System:

1. Release the latch and open the front door.
2. Press and hold **I/Test** on the UPS until you hear a beep, and then release.
3. If the Power LED is not on, press the Power button on the computer.
4. Close and latch the front door. The CR System will not initialize unless the front door is closed.
5. Wait for the CR system to initialize. The Login screen appears.

To shut down the Max CR/CR 975 System:

1. At the **Main Menu**, touch **Quick Menu**. Touch **System Shutdown**.
2. Select **Shutdown/Power Off** to remove power from the system.
3. Select **OK**.

For additional information on these systems, go to

www.carestream.com/documentation

or call Carestream Health, Inc. in your region.

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© Carestream Health, Inc. 2012
ou contacte a Carestream Health, Inc. na sua região.